FR ODM 5®

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CE0459

EN ODM 5®

Preservative-free hyperosmolar ophthalmic solution

PRESENTATION:

Preservative-free 10 ml multi-dose bottle.

COMPOSITION:

Sodium chloride 5%, Sodium hyaluronate 0.15%. Other ingredients: Purified water, trisodium citrate dihydrate, citric acid monohydrate, hydroxypropylmethylcellulose.

ODM 5® contains sodium hyaluronate. This component is naturally present in the eye, and its lubricating properties help protect the cornea.

WHEN SHOULD ODM 5® BE USED?

ODM 5® is recommended for the reduction of corneal oedema: oedema caused by corneal dystrophies, post-traumatic oedema and post-surgical oedema. Corneal oedema is characterised by an excess of fluid in the cornea. ODM 5®, a hypersaline solution attracts this excess fluid away from the cornea by an osmotic effect, enabling the cornea to regain a normal state of hydration for a brief time. In this way ODM 5® participates in improving your quality of vision.

HOW SHOULD ODM 5® BE USED?

- Wash your hands carefully before instilling the product.
- Instil 1 to 2 drops 4 times a day as recommended by your doctor. Apply the drops into the lower conjunctival sac of the eye, looking upwards and pulling the lower lid gently downwards.
- After instillation, keep the eye closed for 1 to 2 minutes.
- Avoid contact between the tip of the bottle and the eye or eyelids.
- Does not require rinsing after use.
- OCULAR ROUTE.

- When using for the first time, check that the tamper-proof ring is intact, then break the seal.
- Remove the protective cap by pulling upwards.

- Turn the bottle in an upside down position above your eye. Hold the bottle in the middle using your index finger and thumb.
- With both fingers, smoothly increase squeezing force until a drop builds up at the tip and falls off. Instill 1 or 2 drops into the eye.
- Replace the protective cap after every use, ensuring the tip is free from any residual drops; if present, shake any off with a sharp wrist movement.

WARNINGS:

- Do not swallow.
- Do not use if bottle is damaged (risk of contamination).
- Do not use after expiry date printed on the bottle.
- Use within 3 months of opening (risk of contamination and loss of product performance).
- Store at room temperature (between +2 and 25°C).
- Carefully close bottle after each use, ensuring the tip is free from any residual drops (risk of contamination).
- Avoid touching the tip of the bottle with your hands, your eye or your eyelid (risk of microbial contamination).
- Avoid touching or drying the tip with fingers or a non-sterile tissue (risk of microbial contamination).
- If your doctor has prescribed additional eye drops, wait at least 15 minutes between applications.
- Keep out of reach and sight of young children.

CONTRAINDICATIONS:

- Hypersensitivity to one of the components.
- Wearers of contact lenses (ODM5® is not compatible with contact lenses).

ADVERSE EFFECTS:

- In some cases transient disorders such as a slight sensation of burning or smarting and momentary blurring of the vision may occur during instillation.
- If there is a persistent adverse effect or one not described in this leaflet, stop using the product and consult your doctor.

Medical device compliant with the requirements of Directive 93/42/EEC amended.

Date marking affixed: October 2013.

Date of last revision of the text: November 2018.

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